Compressor Nebuliser

- 1 Piston compressor
- 2 On/Off switch
- 3 Air filter compartment
- 4 Air filter
- 5 Nebuliser
 - Vaporiser head
- 6 Air hose
- 7 Mouthpiece
- 8 Adult air mask and make sure that it encloses the mouth and nose area completely
- 9 Child face mask
- 10 Replacing air filter
- 11 Assembling nebuliser kit
- 12 Nose piece
- 13 Air filter

Dear Customer,
This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. This device has been specifically designed to make treatment more pleasant for children, whilst ensuring all the quality and safety standards prescribed by law. If you have any questions, problems or want to order spare parts please contact your local Microlife-Customer Service. Your dealer or pharmacy will be able to give you the correct Spare Parts Microlife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of available information on our products.

Stay healthy – Microlife AG!

1. Explanation of Symbols

- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Read the instructions carefully before using this device.
- Type BF applied part
- Class II equipment
- Serial number
- Reference number
- Manufacturer
- ON
- OFF
- IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

Cleaning and disinfecting of the accessories
Follow carefully the cleaning and disinfecting instructions of the accessories as they are very important to the performance of the device and success of the therapy.

Before and after each treatment
Disassemble the nebuliser (2) by turning the top counter-clockwise and remove the nebuliser from the device. Wash the components of the disassembled nebuliser, the mouthpiece (7) and the nose piece (12) by using tap water; dip in boiling water for 5 minutes. Reassemble the nebuliser components and connect it to the air hose correctly, switch the device on and let it work for 10-15 minutes.

Wash masks and air tube with warm water.
Only use cold disinfecting liquids following the manufacturer's instructions.

Do not boil nor autoclave the air tube and masks.

5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact Microlife-Service (see below).

Replacement of the nebuliser
Replace the nebuliser (2) after a long period of inactivity, in cases where it shows deformations, breakage, or when the manufacturer has issued a recall notice. In any case, we recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

Only use the original filters!
In normal conditions of use, the air filter (13) must be replaced approximately after 500 working hours or after each year. We recommend to replace the air filter after (12) months if the filter shows a grey, brown colour or is wet, replace it. Extract the filter and replace it with a new one.

Do not try to clean the filter for reusing it.
The air filter must be replaced or maintained while in use with a patient.
Only use original filters! Do not use the device without filter!

6. Malfunctions and Actions to take
The nebuliser cannot be switched on.
Ensure the power lead (2) is correctly plugged into the socket.
Ensure the ON/OFF switch (3) is in the position «I».

Make sure that the device has been operating within operating limits indicated in this manual (30 min On / 30 min Off).
The nebuliser functions poorly or not at all.
Ensure the air hose (6) is correctly connected at both ends.
Ensure the air hose is not squashed, bent, dirty or blocked. If necessary, replace with a new one.

Ensure the nebuliser (5) is fully assembled and the power cord, keep the plug accessible when the device is in use.
The power plug is a separate element from the power cord, keep the plug accessible when the device is in use.

Before plugging in the device, make sure that the electrical plug is in good condition and is at the bottom of the unit, corresponds to the mains rating. In case the power plug provided with the device does not fit your wall socket, contact qualified personnel for replacement plug with that of date of purchase or later. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use adapters complying with safety regulations, paying attention that they do not exceed the maximum power limits, indicated on adapters and extension cables.
Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not in operation.

The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or objects, for which the manufacturer cannot be held responsible.
Do not replace the power lead of this device. In case of a power lead damage, contact a technical service centre authorized by the manufacturer for its replacement.
The power supply cord should always be fully unwound in order to prevent dangerous overheating. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.

Only use the medication prescribed for you by your doctor and follow your doctor's instructions in relation to dosage, duration and frequency of the therapy.
Depending on the pathology, only use the treatment that is recommended by your doctor.
Only use the nose piece if expressly indicated by your doctor, paying special attention NEVER to introduce the nebulisations into the nose, but only positioning them as close as possible.
Check in the medicine instruction leaflet for possible contraindications for use with common aerosol therapy systems.
Do not position the equipment so that it is difficult to observe the disconnection device.
Do not use the equipment if you do not use the same accessories for more than one person.
Never bend the nebuliser over 60°.

Do not use this device close to electromagnetic fields such as mobile telephones or radio installations. Keep a minimum distance of 3.3 m from such devices when using this device.
Ensure that children do not use this device unsupervised. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.
Use of this device is not intended as a substitute for a consultation with your doctor.

This device is not a toy! Keep your child out of reach of children and do not leave children alone during treatment.

3. Preparation and Usage of This Device

Prior to using the device for the first time, we recommend reading it as described in the section «Cleaning and Disinfecting».

1. Assemble the nebuliser kit (1). Ensure that all parts are complete.

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